



Glooko[®] Uploader

Instructions for use – for Glooko users



Contents

Find your device	4-5
General information	6
Product description.....	6
Intended use.....	6
Supported devices.....	6
Warnings.....	6
Contraindications	6
Instructions for use	7
Patients	8
Sign into your MyGlooko Dashboard at https://my.glooko.com	8
Download the software installation file	8
Locate the installation file and run the file.....	8
Health care providers.....	9
Request a link to download the Glooko Uploader software.....	9
Download the software installation file	9
Locate the installation file and run the file.....	9
Patients & Health care providers.....	10
Open the Glooko Uploader software.....	10
Sign into your Glooko account.....	10
Features in Glooko Uploader	11
Upload diabetes data	12
View diabetes data.....	13
Compatible devices and transmission methods	14

Find your device



Glucose meter



CGM



Insulin pump

Abbott

	FreeStyle Freedom Lite.....	14
	FreeStyle InsuLinx.....	14
	FreeStyle Lite.....	14
	FreeStyle Optium Neo.....	14
	FreeStyle Precision Neo.....	14
	Precision Xtra.....	14
	Precision Xceed.....	14

	FreeStyle Libre Reader.....	14
	FreeStyle Libre 2 Reader.....	14
	FreeStyle Libre 3 Reader.....	14

Acon Laboratories

	On Call® Express.....	14
	On Call® Express II.....	14
	On Call® Express Voice.....	14
	On Call® Extra.....	14
	On Call® Extra Voice.....	14
	On Call® Sure.....	14
	On Call® Sure Sync.....	14

Activmed

	GlucoCheck GOLD (TD-4289).....	14
--	--------------------------------	----

AgaMatrix

	WaveSense Jazz™.....	14
	Ally BGK.....	14
	Ally GGM.....	14

Animas

	OneTouch Ping (meter).....	14
--	----------------------------	----

Animas

	OneTouch Ping (pump).....	20
	Vibe.....	20

Arkray

	GlucoCard Shine Connex.....	15
	GlucoCard Shine Express.....	15
	GlucoCard Shine XL.....	15

Ascensia (Bayer)

	Breeze 2.....	15
	Contour.....	15
	Contour Care.....	15
	Contour Fit.....	15
	Contour Link.....	15
	Contour Next.....	15
	Contour Next (Bluetooth).....	15
	Contour Next Gen.....	15
	Contour Next One.....	15
	Contour Plus.....	15
	Contour Plus Blue.....	15
	Contour Plus Elite.....	15
	Contour Plus One.....	15
	Contour next EZ.....	15
	Contour next Link.....	15
	Contour next Link 2.4.....	15
	Contour next USB.....	15
	Contour TS.....	15
	Contour XT.....	15
	Contour USB.....	15

Beurer

	GL 44.....	15
	GL 48.....	15
	GL 49.....	15
	GL 60.....	15

Bionime

	Rightest GM700SB.....	15
--	-----------------------	----

Biosys

	Aria.....	15
--	-----------	----

Dexcom

	G4 Platinum.....	19
	G4 Platinum with Share.....	19
	G5.....	19
	Touchscreen Receiver (G5/G6).....	19
	G6 Receiver.....	19
	G7 Receiver.....	19
	One Receiver.....	19
	One+ Receiver.....	19

Diatesse

	XPER (TD-4289B).....	15
--	----------------------	----

Ekoweb

	GlucoX (TD-4183).....	16
	GlucoX Pro (TD-4183).....	16

Embrace

	TALK.....	16
--	-----------	----

Exacta

	Glance.....	16
--	-------------	----

Equil/Wellion

	MICRO-pump.....	21
--	-----------------	----

ForaCare

	Diamond PRIMA.....	16
--	--------------------	----

GlucorX

	Nexus.....	16
	Nexus Mini.....	16
	Q.....	16

GlucoSmart

	Salsa.....	16
	Salsa Connect.....	16

GlucoX

	TD-4183.....	16
--	--------------	----

iCare


	PalmDoc II.....	16
--	-----------------	----


i-SENS

	alphacheck professional.....	16
	CareSens BGM/TEE2.....	16
	CareSens Dual.....	16
	CareSens H Beat.....	16
	CareSens N.....	16
	CareSens N Feliz.....	16
	CareSens N POP.....	16
	CareSens N Premier.....	16
	CareSens N Premier Carelight.....	16
	CareSens N Voice.....	16
	Caresens Pro.....	16
	CareSens S Fit.....	16
	No Coding Plus.....	16
	TEE2+.....	16

Infopia	
 Finetest Lite	16


Insulet	
 Omnipod®	21
 Omnipod® Dash™ System	21

Intuity	
 POGO® Automatic™	17

LifeScan	
 OneTouch Select Plus	17
OneTouch Ultra.....	17
OneTouch Ultra Plus Reflect.....	17
OneTouch Ultra2	17
OneTouch UltraEasy.....	17
OneTouch UltraMini	17
OneTouch Verio	17
OneTouch Verio Flex.....	17
OneTouch Verio IQ.....	17
OneTouch Verio Reflect.....	17
OneTouch Verio Sync.....	17

Medtronic	
 630G.....	21
 640G.....	21
 670G.....	21

Menarini	
 GlucoFix Tech	17
GlucMen areo.....	17
GlucMen areo 2K.....	17
GlucMen LX2.....	17


Nipro	
 4SURE Smart	17
 4SURE Smart Duo	17

Palmdoc	
 1	17
 Smart.....	17


Prodigy	
 AutoCode.....	17

ReliOn	
 Platinum.....	18
Premier Blu.....	18
Premier Classic.....	18
Premier Voice.....	18



Roche	
 Accu-Chek Active.....	17
Accu-Chek Aviva Connect.....	17
Accu-Chek Aviva Combo.....	19
Accu-Chek Aviva Expert.....	19
Accu-Chek Aviva Nano.....	19
Accu-Chek Aviva Plus Black	19
Accu-Chek Aviva Plus Silver	19
Accu-Chek Compact Plus.....	19
Accu-Chek Guide.....	17
Accu-Chek Guide Me.....	17
Accu-Chek Instant.....	17
Accu-Chek Mobile (USB)	17
Accu-Chek Nano.....	19
Accu-Chek Performa	19
Accu-Chek Performa Nano	19

Roche	
 Accu-Chek Aviva Insight	22
 Accu-Chek Performa Insight.....	22
Accu-Chek Aviva Solo.....	21
Accu-Chek Guide Solo	21
Accu-Chek Spirit Combo	22

Sanofi	
 BGStar	18

Taidoc	
 TD-4141	18
TD-4183	18
TD-4255	18
TD-4257(v1)	18
TD-4257(v2)	18
TD-4280	18
TD-4285	18
TD-4289	18

Tandem	
 t:flex.....	21
t:slim.....	21
t:slim G4.....	21
t:slim X2	21


Terumo	
 Medisafe Fit Smile	20
 Medisafe WITH.....	20

ViCentra	
 Kaleido.....	21

Trividia (Nipro)	
 TRUE METRIX	17
TRUE METRIX AIR.....	17
TRUResult.....	17
TRUEyou mini.....	17

Wellion	
 Calla Mini.....	18
Galileo Compact.....	18
Leonardo Compact	18

XPER Technology	
 A1 (TD-4183).....	18

Ypsomed	
 Mylife Aveo.....	18

General information

Product description

Glooko Uploader® is indicated for use by individuals or health care professionals in the home or health care facilities to transfer pre-defined data from home monitoring devices to a server database. The device can be used as a stand-alone product or in combination with Glooko System.

Intended use

The Glooko System is a diabetes management platform that can be used in-clinic or remotely by Health Care Professionals and patients. It allows viewing of patient data from compatible medical devices and other health monitoring devices. It also allows Health Care Professionals to support the treatment of diabetes through analysis and monitoring of a patient's diabetes therapy and care program.

Supported devices

To get started, individuals will need:

1. A compatible blood glucose meter, insulin pump, or continuous glucose monitor (CGM);
2. A PC or Mac computer with one of the following Operating Systems (OS):
 - Windows 10 (32-bit and 64-bit)
 - Windows 11 (32-bit and 64-bit)
 - macOS 14.0: Sonoma
 - macOS 15.0: Sequoia
 - macOS 26.0: Tahoe
3. The Glooko Uploader software;
4. A Glooko account; and
5. The syncing or device download cable that came with the supported diabetes device.

Warning!

Glooko does not measure, interpret, or make decisions on the data it conveys nor is it intended to provide automated treatment decisions or be used as a substitute for professional judgment. All medical diagnosis and treatment are to be performed under the supervision and oversight of an appropriate healthcare provider.

If you have concerns with your current diabetes diagnosis and treatment, please contact your healthcare provider. In the event where the Glooko System is malfunctioning, please discontinue use of the product and contact Glooko Support.

Contraindications


None.

Instructions for use – Glooko Uploader

You will need to create a Glooko account to use the Glooko Uploader. The data uploaded using the Glooko Uploader will synchronize to your Glooko account, which can be accessed with your Glooko login information. Internet (WiFi or cellular data network) is necessary for your data to synchronize to the Glooko server.

Patients:

- Learn how to install and upload data using Glooko Uploader on the next page.

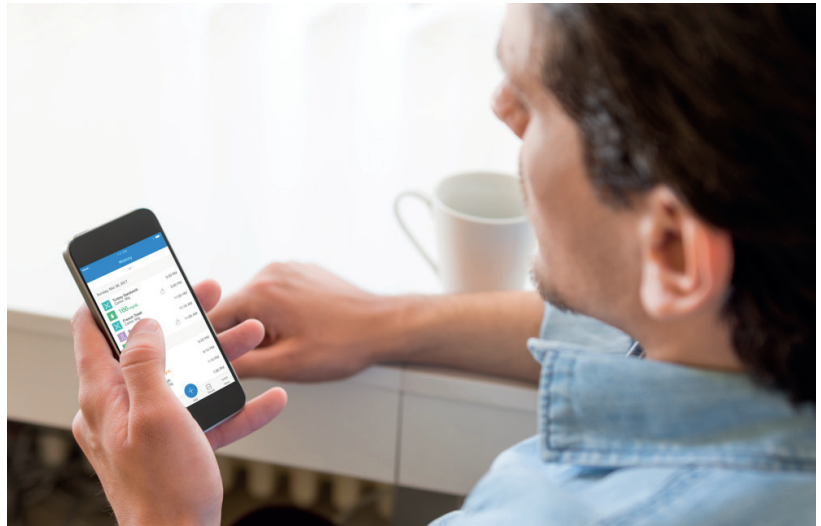

Health Care Providers:

- Learn how to install and upload data using Glooko Uploader on page 9.



Patients

In order to access the MyGlooko Dashboard and download the Glooko Uploader software, you will need to have a Glooko account. You can sign up for Glooko using the Glooko mobile app for iOS or Android. You can also contact **Glooko Support** (<https://support.glooko.com>) for assistance creating an account. Once you have created a Glooko account, please follow the instructions below.

1 Log into your MyGlooko Dashboard at <https://my.glooko.com>

- Enter your Email Address and Password.
- Click **Log In**.


GET HELP SIGNING INTO YOUR GLOOKO ACCOUNT

• FORGOT PASSWORD:

If you are unable to remember your password to Login, click **Forgot Password** to receive a password reset email. Follow instructions on the screen. You can also contact Glooko Support for assistance.

• UNLOCK YOUR ACCOUNT:

If you have entered your password incorrectly five times, your account will become locked. An email with unlock instructions will automatically be sent to your account email address. However, if you did not receive an email, click **Didn't receive unlock instructions** - you will be redirected to the Unlock page. Follow instructions on the screen. You can also contact Glooko Support for assistance.

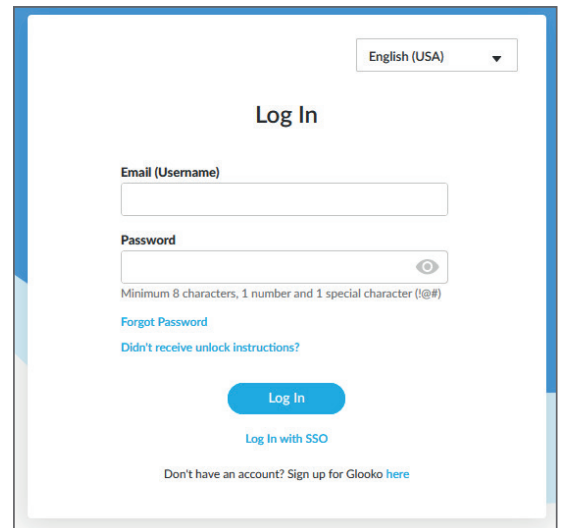

2 Download the software installation file

After logging into your MyGlooko Dashboard, you will need to download the Installation File.

- Click **Settings**.
- Scroll to the bottom of the page and click **Get Glooko Uploader**.
- You will be taken to a screen with buttons to install the Glooko Uploader on a Mac or PC. Click either the Mac or PC button to download the Installation File.



3 Locate the installation file and run the file

Depending on the web browser and operating system, you may need to search for the downloaded installation file or you may see a pop up window asking you to run the file.

- If you are using a **PC**, look in your recent downloads folder
- If you are using a **Mac**, look in the downloads folder in Finder

Once you've located the file, double-click on it and follow the installation instructions on the screen. You will be required to restart your computer at the end of the installation process.

NOTE: For FreeStyle Libre users in Sweden, Norway and the US, it's important to select that you are using a FreeStyle Libre during the installation process. This will install the additional FreeStyle Libre DataHub required for these uploads.

Health care providers

In order to use the Glooko Uploader software, you will need a Glooko Provider account and a subscription that supports the Glooko Uploader. You will also need a specific link to download the Glooko Uploader. Please contact Glooko Support (<https://support.glooko.com>) for assistance.

1 Request a link to download the Glooko Uploader software

Health Care Providers must request a link to download the Glooko Uploader software.

- Please contact Glooko Support (<https://support.glooko.com>) to request a link to download the Glooko Uploader software.

2 Download the software installation file

- When you receive the link to download the software from Glooko, click the link in your email. You will be taken to a screen with buttons to install the Glooko Uploader on a Mac or PC. Click either the Mac or PC button to download the Installation File.



3 Locate the installation file and run the file

Depending on the web browser and operating system, you may need to search for the downloaded installation file or you may see a pop up window asking you to run the file.

- If you are using a **PC**, look in your recent downloads folder
- If you are using a **Mac**, look in the downloads folder in Finder

Once you've located the file, double-click on it and follow the installation instructions on the screen. You will be required to restart your computer at the end of the installation process.

NOTE: For FreeStyle Libre users in Sweden, Norway and the US, it's important to select that you are using a FreeStyle Libre during the installation process. This will install the additional FreeStyle Libre DataHub required for these uploads.

Patients & Health care providers

After installing the Glooko Uploader and restarting your computer, locate the Glooko Uploader icon either on your Desktop (PC) or in your Applications folder (Mac).

4 Open the Glooko Uploader software

- **Locate** the Glooko Uploader icon on your Desktop (PC) or in your Applications Folder (Mac).
- **Double-click** on the Glooko Uploader icon to start the program.

NOTE: Make sure you have restarted your computer before starting the Glooko Uploader for the first time.



5 Sign into your Glooko account


Once the Glooko Uploader program opens, you will need to sign into your Glooko Account.

- Enter your **Email or Username**
- Enter your **Password**
- Click **Sign In**.

NOTE: If you've activated Two-Factor Authentication (2FA) on your Glooko account, you'll need to use both your password and a one-time password (OTP) to complete the login process.

GET HELP SIGNING INTO YOUR GLOOKO ACCOUNT

If you are unable to remember your password to Login, click **Forgot Password** to receive a password reset email. You can also contact Glooko Support for assistance.

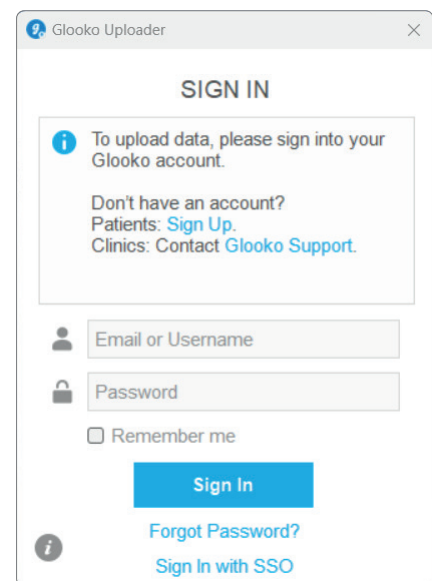

SINGLE SIGN-ON (SSO) FOR HEALTH CARE PROVIDERS (HCPS)


The Single Sign-On (SSO) feature allows Health Care Professionals to log in using their clinic's existing secure authentication service (Identity Provider (IdP)).

1. Click the "Sign in with SSO" option at the bottom of the sign-in window.
2. You will be directed to the Glooko Web where you need to enter the email address associated with your Glooko account.
3. Glooko will communicate with your clinic's configured system to identify your IdP and you will be redirected to your IdP's login page.
4. Log in using your usual clinic credentials for that Identity Provider.
5. Once authenticated by your clinic's system, you will be directed back to the Glooko Uploader screen to proceed with device upload.


Patients & Health care providers


6 Features in Glooko Uploader

LOG OUT YOUR ACCOUNT


- You can log out of your Glooko account by clicking the Profile button  in the bottom right-hand corner of the Glooko Uploader.
- Click **Log out**


VIEW YOUR PROFILE INFORMATION

- You can view your profile information by clicking the Profile button  in the bottom right-hand corner of the Glooko Uploader.
- This popup includes:
 - o Your **Name**
 - o Your **Primary Authorized Site** (Health Care Provider) and the associated **ProConnect Code**
 - o Your **Email address**
- Click **Log Out**.

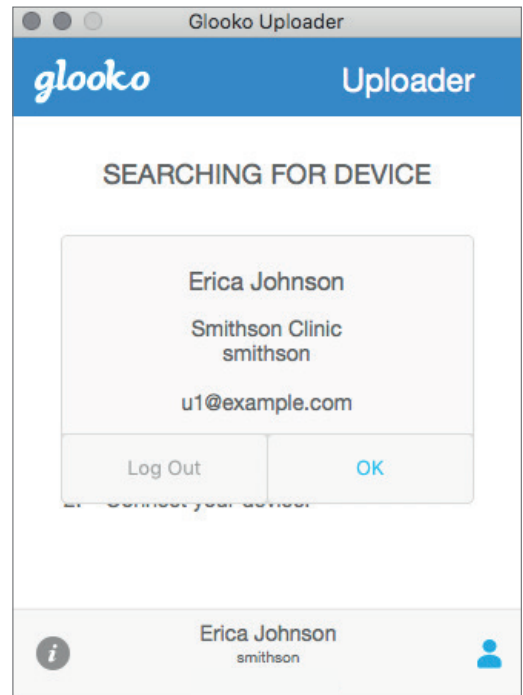

SEE ADDITIONAL INFORMATION ABOUT GLOOKO UPLOADER

- You can view additional information by clicking the Info button  in the bottom left-hand corner of the Glooko Uploader.
- This popup includes product support information, product description, and the Intended Use statement.

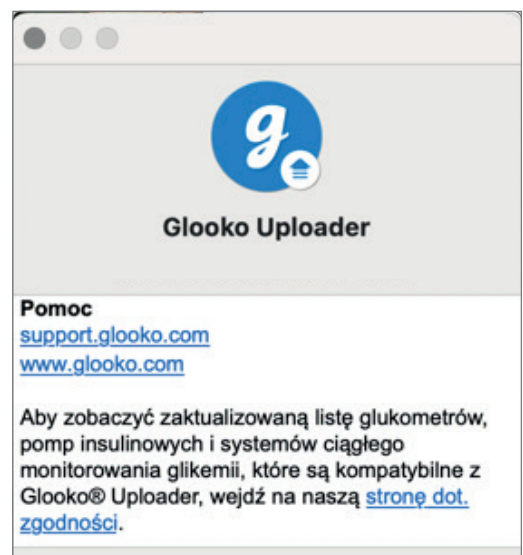

Patients & Health care providers


7 Upload diabetes data

Once you have signed into the Glooko Uploader, you can upload your diabetes device data.

CONNECT THE CABLE FOR YOUR DIABETES DEVICE TO THE COMPUTER

After logging in, Glooko Uploader will start searching for a device.

- Connect the appropriate USB cable or infrared (IR) cable for the diabetes device to the computer.
- Once the cable is recognized, wait a few seconds for the cable drivers to be recognized before connecting your device.

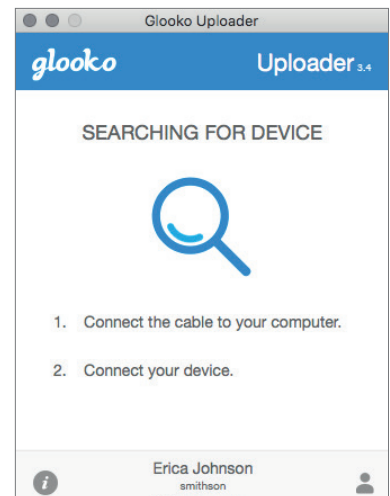


CONNECT THE DIABETES DEVICE TO THE CABLE

- After connecting the cable to the computer, connect the diabetes device to the cable.

NOTE: Make sure you use the cable that came with your diabetes device. If you need to get a new cable, please contact the manufacturer of your diabetes device.

- After connecting the diabetes device to your computer using the cable that came with the device, your device will begin uploading data using the Glooko Uploader.
- If you receive an error, please follow the instructions on the screen to resolve the error. If you continue to experience difficulties, please contact Glooko Support for assistance.

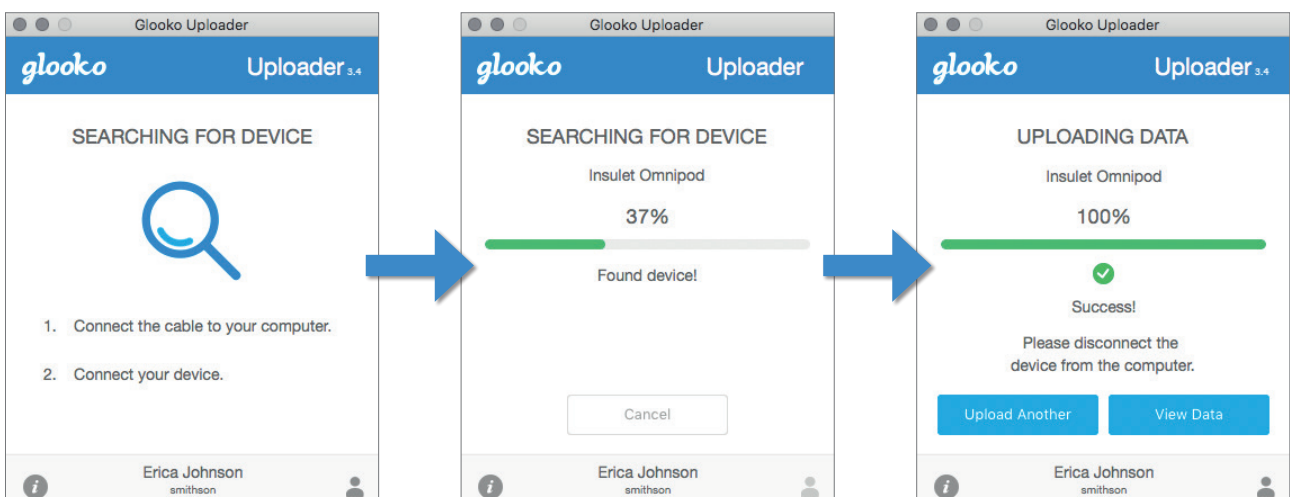

Patients & Health care providers

8 View diabetes data

After uploading your diabetes device data using the Glooko Uploader, you can view the data at <https://my.glooko.com> by logging into your Glooko account.

- **Patients:**

To learn more about the MyGlooko Web Dashboard, please refer to the [Glooko for Personal Use - Quick Start Guide](#) at <https://support.glooko.com>


- **Health Care Providers:**

To learn more about the Population Tracker, please refer to the: [Glooko for Clinics - Quick Start Guide](#) at <https://support.glooko.com>



Compatible devices and transmission methods

Below are instructions on how to upload diabetes data to Glooko using Glooko® Uploader. Unless explicit upload instructions are provided for the device, you simply need to connect it to the PC/Mac with its specific USB cable and it will upload automatically.

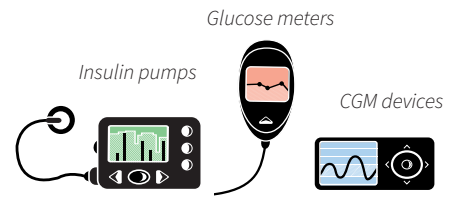

i Please note that some of the diabetes devices may not be available in your market.

Blood glucose meters – connected with USB cable

Abbott

FreeStyle Freedom Lite | FreeStyle Lite

Abbott FreeStyle USB (2,5 mm)




FreeStyle InsuLinx | FreeStyle Optium Neo
FreeStyle Precision Neo

Micro USB



Precision Xceed | Precision Xtra

Abbott Xceed USB

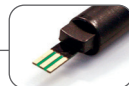

FreeStyle Libre Reader | FreeStyle Libre 3 Reader
FreeStyle Libre 2 Reader

Micro USB



Compatibility is limited to Sweden, Norway, and the US by Abbott.

NOTE: During the installation process of the Glooko Uploader software, be sure to select that you're using a FreeStyle Libre. This will install the additional FreeStyle Libre DataHub required for these uploads.

Acon Laboratories

On Call® Express | On Call® Extra Voice
On Call® Express II | On Call® Sure
On Call® Express Voice | On Call® Sure Sync
On Call® Extra

On Call USB (2,5 mm)



Activmed

GlucoCheck GOLD (TD-4289)

Micro USB



Agamatrix

WaveSense Jazz

Agamatrix USB



Ally BGK

Micro USB



Ally BGM

i-Sens USB (2,5 mm)



Animas


OneTouch Ping (meter)

Mini USB




1. Connect to your computer.
2. Press the meter's power button.



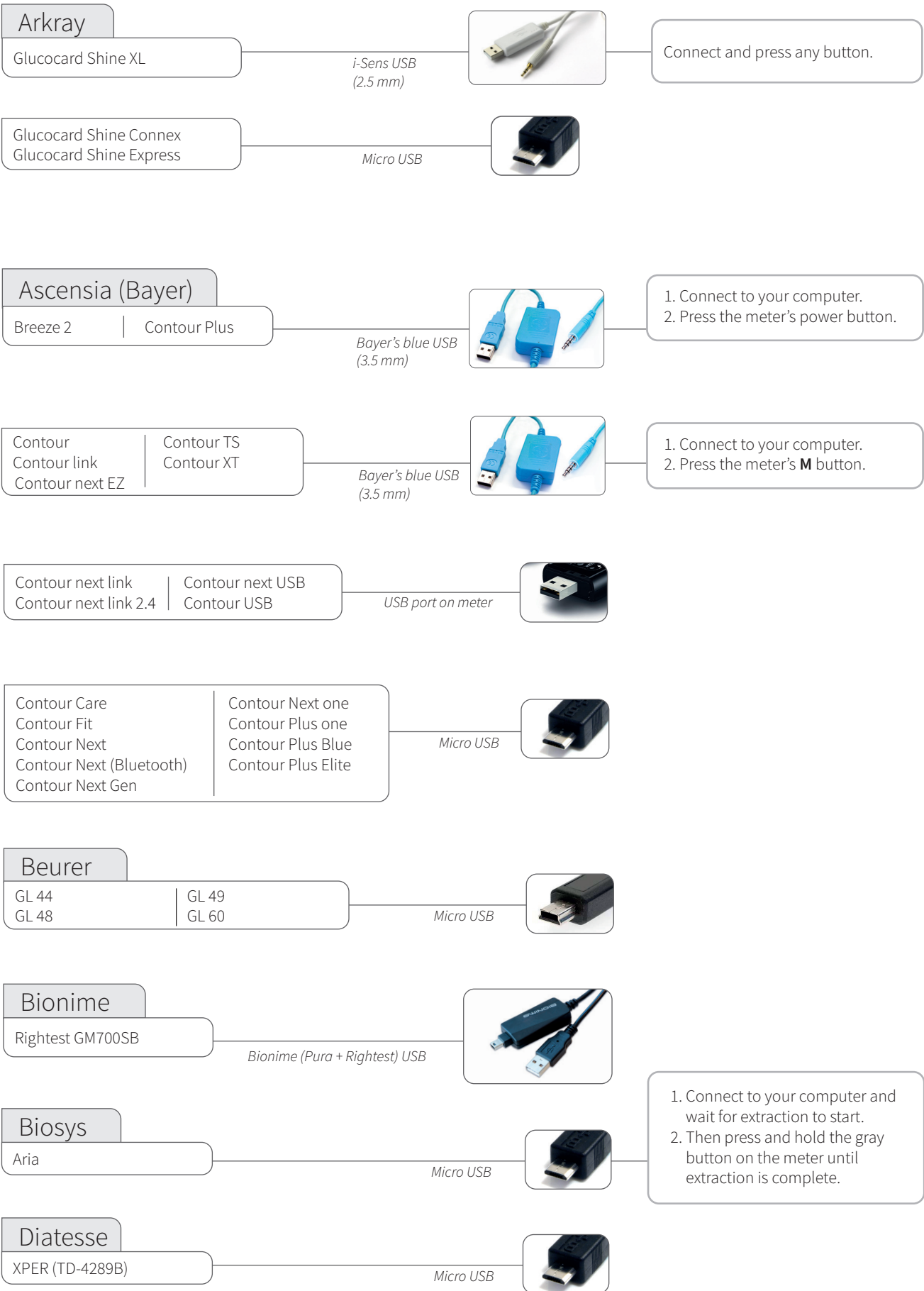



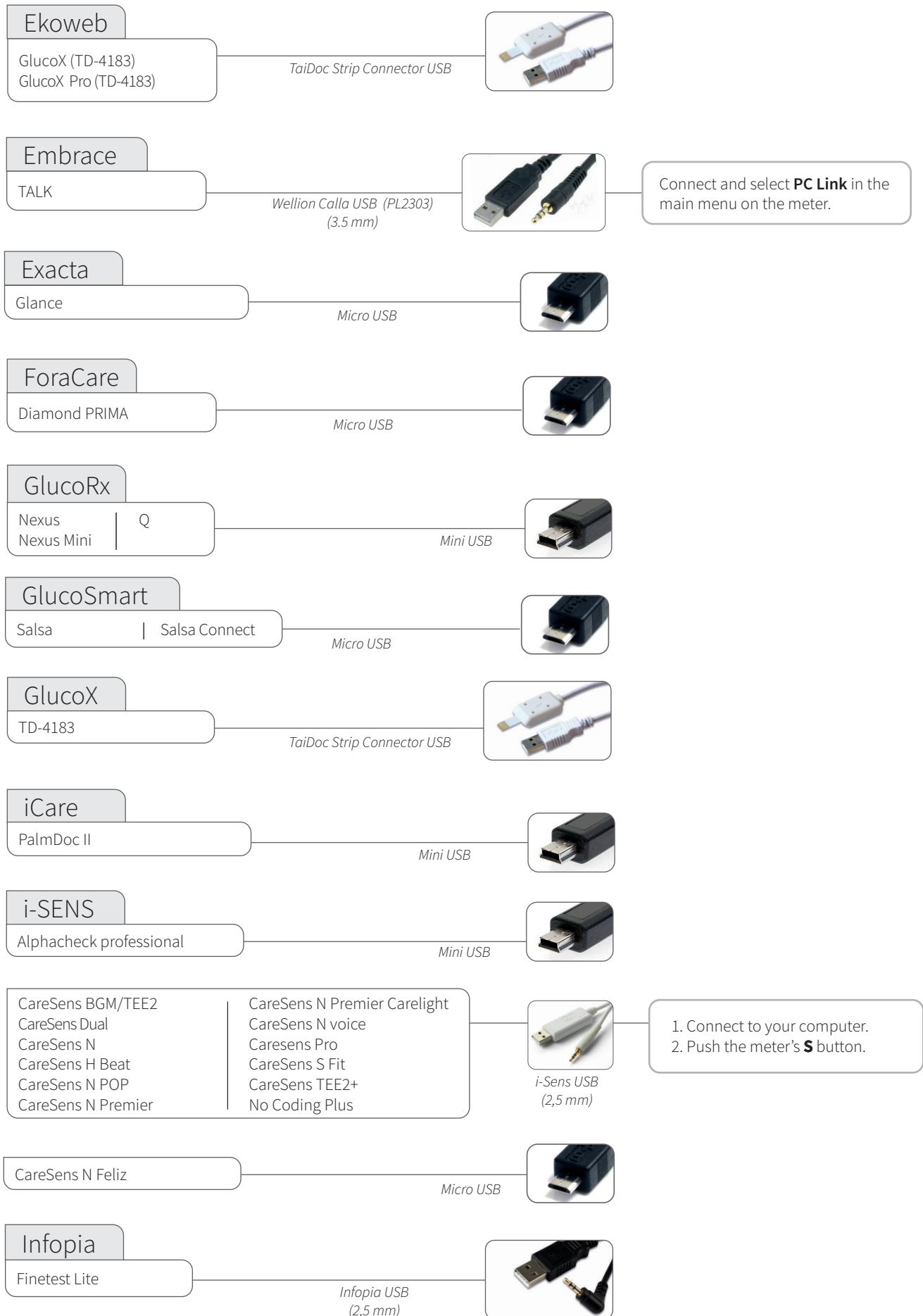





Blood glucose meters – connected with USB cable




Blood glucose meters – connected with USB cable



Blood glucose meters – connected with USB cable

<p>Intuity POGO® Automatic™</p>	<p>Micro USB</p>		
<p>LifeScan OneTouch Ultra OneTouch UltraEasy OneTouch Ultra2 OneTouch UltraMini</p>	<p>LifeScan USB (3.5 mm)</p>		<p>Connect and press any button.</p>
<p>OneTouch Select Plus OneTouch Verio Flex OneTouch Ultra Plus Reflect OneTouch Verio Reflect OneTouch Verio</p>	<p>Micro USB</p>		
<p>OneTouch Verio IQ OneTouch Verio Sync</p>	<p>Mini USB</p>		
<p>Menarini GlucoMen areo GlucoFix Tech GlucoMen areo 2K</p>	<p>Menarini GlucoMen areo USB (2,5 mm)</p>		
<p>GlucoMen LX2</p>	<p>Menarini blue USB (3.5 mm)</p>		<p>If the meter displays: E-3, disconnect it wait until the display is off, then connect the meter again.</p>
<p>Nipro 4SURE Smart 4SURE Smart Duo</p>	<p>Micro USB</p>		
<p>Trividia (Nipro) TRUEresult TRUE METRIX AIR TRUE METRIX TRUEyou mini</p>	<p>Nipro dock USB</p>		<p>Connect and place the meter in the docking station.</p>
<p>Palmdoc 1</p>	<p>Palmdoc 1 USB (2,5 mm)</p>		
<p>Smart</p>	<p>Micro USB</p>		
<p>Prodigy AutoCode</p>	<p>Mini USB</p>		
<p>Roche Accu-Chek Active Guide Me Aviva Connect Instant Guide Mobile (USB)</p>	<p>Micro USB</p>		<ol style="list-style-type: none"> 1. Connect to your computer. 2. Upload should start within 20 seconds after connecting the cable. If not, go to the main menu on the meter. Go to My data > PC analysis > Data Transfer, and connect the meter to the USB cable again. <i>(Please do not hold down the two arrow buttons.)</i>

Blood glucose meters – connected with USB cable

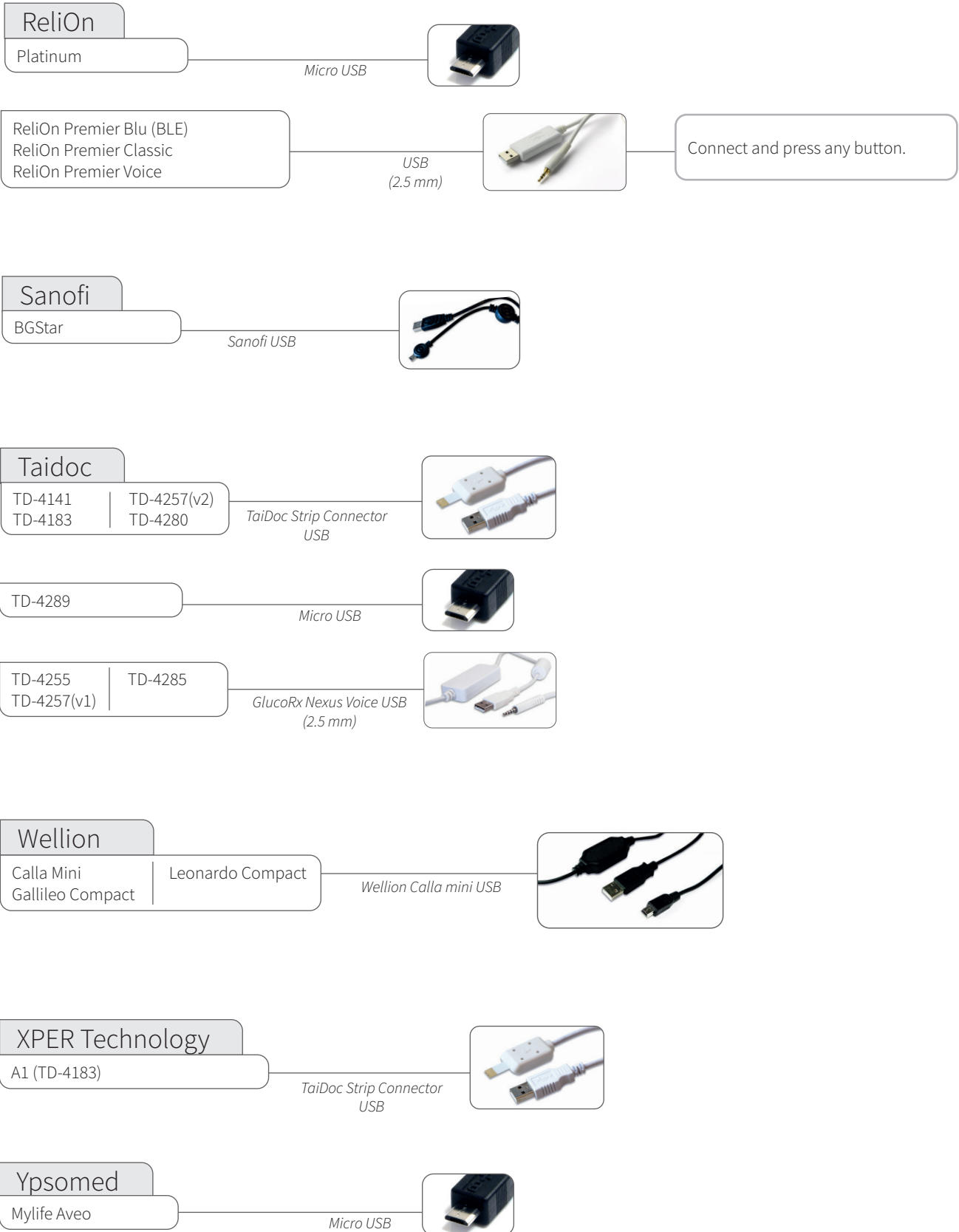

Blood glucose meters – with infrared connection



Roche (IR)

Accu-Chek Aviva Nano
Accu-Chek Aviva Plus Black
Accu-Chek Aviva Plus Silver
Accu-Chek Nano
Accu-Chek Performa
Accu-Chek Performa Nano



ACTISYS Li, Li2 or Accu-Chek 360°

1. Connect IR cable to your computer.
2. With the meter off, activate IR transmission by pushing and holding the two arrow buttons until two arrows appear in the display.

Accu-Chek Compact Plus



ACTISYS Li, Li2 or Accu-Chek 360°

1. Connect IR cable to your computer.
2. Activate IR transmission by pushing and holding the buttons **S** and **M** below the display until two arrows appear in the display.

Accu-Chek Aviva Combo
Accu-Chek Aviva Expert



ACTISYS Li, Li2 or Accu-Chek 360°

1. Connect IR cable to your computer.
2. In the main menu, go to **My data** and select **Data transfer**.

CGMs - connected with USB cable

Dexcom

G4 Platinum
G4 Platinum with Share
G5
Touchscreen Receiver (G5/G6)
G6 Receiver
G7 Receiver
One Receiver
One+ Receiver

Micro USB




If the upload doesn't start, disconnect the receiver, select Shutdown in the menu to turn the receiver off. Turn the receiver on again and connect it to the computer.

Blood glucose meters and Insulin pumps - with NFC connection

Terumo


Medisafe Fit Smile



1. Connect the NFC plate provided by Terumo to your computer.
2. Place the meter on top of the NFC plate.

Please note that the transmission symbol on the back of the remote control needs to be aligned with the NFC symbol on the NFC Plate.

Medisafe WITH




1. Connect the NFC plate provided by Terumo to your computer.
2. Make sure the main menu is showing on the remote control for the pump.
3. Place the remote control on top of the NFC plate.

Please note that the transmission symbol on the back of the remote control needs to be aligned with the NFC symbol on the NFC Plate.

Insulin pumps – with infrared connection

Animas (IR)

OneTouch Ping Vibe








1. Suspend the pump. (MAIN MENU, scroll to **Suspnd/Resum** and press **OK**; **Suspend** is highlighted).
2. Press **OK**.
3. Place the pump with its back towards the Actisys IR wireless download cable.
4. When the transmission is finished: Resume pump (MAIN MENU, scroll to **Suspnd/Resum** and press **OK**; **Resume** is highlighted). Press **OK**.

ACTiSYS Li or Li2

Note! If you are using Windows 8 or 10, you need to use the ACTiSYS Li2 cable.

Compatible infrared cables


<p>ACTiSYS - Li cable</p>  <p>Compatible operating systems: macOS X: 10.14–12.0 Windows: N/A</p>	<p>ACTiSYS - Li2 cable</p>  <p>Compatible operating systems: macOS X: 10.14–12.0 Windows: 8, 10</p>	<p>Accu-Chek - 360° cable (Realtyme)</p>  <p>Compatible operating systems: macOS X: 10.14–12.0 Windows: 8, 10</p>	<p>Roche Accu-Chek Smart Pix 2 cable</p>  <p>Compatible operating systems: macOS X: 10.14–12.0 Windows: 8, 10</p>
---	--	---	--

 **Missing a cable?**
Please contact the manufacturer of your specific diabetes device.

Insulin pumps – connected with USB cable

Equil/Wellion
MICRO-pump

Micro USB



Insulet
Omnipod® System

Mini USB



Omnipod® Dash™ System

Micro USB



1. Connect and press the PDM's power button.
2. Unlock the PDM and tap 'Export' on the screen.

Medtronic

630G	670G
640G	

USB port on meter

1. Connect a Contour Next Link 2.4 meter to one of your computer's USB ports.
2. Follow the on-screen instructions presented in your Glooko Uploader to upload data from your Medtronic pump.

Roche

Accu-Chek Aviva Solo
Accu-Chek Guide Solo

Micro USB



1. Open the Main Menu on the hand unit for the pump.
2. Select USB in the Main Menu.
3. Connect the hand unit to the computer with the Micro USB cable.

Tandem


t:flex	t:slim G4
t:slim	t:slim X2

Micro USB



Vicentra
Kaleido

Micro USB



Go to the yellow menu and choose data upload.

Insulin pumps - connected with infrared cables

Compatible infrared cables


Roche Accu-Chek Smart Pix cable



Roche Accu-Chek Smart Pix 2 cable



Roche
Accu-Chek Aviva Insight
Accu-Chek Performa Insight

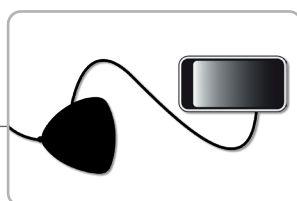

Smart Pix 2

Note! Connect the handset of your pump with a micro USB cable to the Smart Pix 2.

1. Connect the Smart Pix 2 cable to your computer and wait for the blue indicator on the Smart Pix 2 to start flashing slowly.
2. Connect the handset to the Smart Pix 2 with a micro USB cable.
3. In the handset menu select **Connect to PC** and press **OK**.
4. Ensure that the pump is within reach of the handset.
5. The blue indicator on the Smart Pix 2 is on while the data is being transmitted from the device to the Smart Pix 2.
6. The blue indicator turns itself off and both Smart Pix 2 and device emit a sound to signal that the data is being transmitted from the Smart Pix 2 to Glooko Uploader. Do not disconnect the Smart Pix 2 yet!
7. Wait for Glooko Uploader to confirm the upload was successful.

Roche
Accu-Chek Spirit Combo




Smart Pix or Smart Pix 2

Note! Smart Pix software version 3.02 or higher is required when using Smart Pix.

1. Connect the Smart Pix to your computer and wait for the blue indicator on the Smart Pix to start flashing slowly.
2. Put the pump in **Data Transfer** mode (stop the pump and enter the **Data Transfer** menu).
3. Place the pump with its IR eye towards the Smart Pix. The blue indicator signals the data is being transmitted to the Smart Pix.
4. Wait for the blue indicator to start flashing slowly again or stop flashing altogether.
5. The data is now being transmitted from the Smart Pix to Uploader. Do not disconnect the Smart Pix yet!
6. Wait for Uploader to signal the upload was successful.



 Glooko Inc
579 University Avenue
Palo Alto, CA 94301
USA

Support
support@glooko.com
<https://support.glooko.com>